

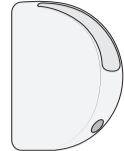






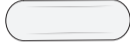




-  3x
-  2x
-  2x
-  6x
-  2x
-  6x
-  6x
-  6x
-  2x
-  2x
-  2x
-  1x



RIDDER | *Assistent*

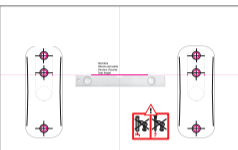

★★★★★ COMFORT



A140101 A140107 A140108 A140110



Art.-Nr.: A1401007
2492024

-  1x
-  1x

RIDDER GmbH
Ridder Straße 1
D-56379 Singhofen
Tel.: +49 (2604) 9541-0
Mail: info@ridder-online.de
www.ridder-online.de

FR
Pensez à donner ou recycler.

Association ou Magasin ou Déchèterie

<https://quefairemesdechets.fr>

RECICLA
PAP21
RACCOLTA DIFFERENZIATA

PAP

1. Attach the long thin bars to the base using screws A and B.

2.1. Rotate the bars 180°.

2.2. Attach the curved parts to the bars.

3. Attach the side rails to the curved parts.

3.1. Attach the side rails to the base.

3.2. Attach the side rails to the base.

4. Use a level to ensure the stretcher is level. Height: ca. 410 - 540mm.

5. Mark the wall with a grid and level.

6. Drill holes in the wall (Ø 8mm).

7. Insert the sawtooth anchors into the wall.

8. Attach the side rails to the wall using screws B.

9. Attach the side rails to the wall using screws B.

10. Attach the side rails to the wall using screws B.

11. Attach the side rails to the wall using screws B.

12. Attach the side rails to the wall using screws A.

13. Final check and warning: Do not use the stretcher as a ladder.